


Harvest Healthcare Ltd  
Sheaf House  
Bradmarsh Way  
Bradmarsh Business Park  
Tel +44(0)1709 377172  
Fax +44(0)1709 377173



## Instructions for Use of HTE001.2 Evacuation Sheet for Active Mattresses

Harvest Healthcare reserves the right to alter or amend  
this document without notice.

Company Accredited to  
ISO 9001 & 13485:2003 Standards

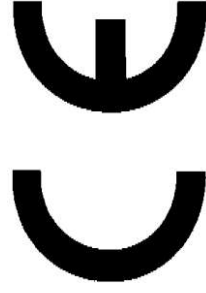

Harvest Healthcare Recommends that the Active Mattress Evacuation Sheet is used in accordance with the following instructions.



### 1. Instructions for Use

The Harvest Active Mattress Evacuation Sheet is to be used in conjunction with an Active mattress system. During the evacuation the straps are secured around the mattress and the patient who is the safely and securely evacuated to safety.

### 2. Maintenance

The Harvest Active Evacuation Sheet is a low maintenance product. Limited inspections may be necessary every so often. These are strictly single use only, warranty is void after use.

All Harvest Evacuation devices are tested to a suspended weight of 320kg for a 10 Minute Period and tested to comply with IEC 60601-1:2006

### 3. Warranty

Harvest healthcare Ltd offers a limited one (1) year products warranty on all defects in workmanship and fabrics. **“this is strictly a single use product”** means that this product should be used for one evacuation only; it can be used as many times as the customer wishes during that evacuation. We recommend that it is checked for wear and tear before each patient is put on it. Once the evacuation is over then this product must be thrown out. Warranty covers manufacturing defects only and is invalidated after single use.

### 4. Fire Ratings

The Harvest Active Evacuation Sheet is fire retardant to Source 5, this is standard for care homes and hospitals, certificates available on request

### 5. Cleaning Recommendations

Harvest Healthcare recommends that the following instructions are followed when cleaning the HTE001.2 Active Evacuation Sheet.

- For light soiling the Evacuation sheet should just be wiped clean using a mild detergent.
- If the evacuation sheet becomes heavily soiled if can be washed in a washing machine, full decontamination procedure available from Harvest Healthcare on request

Extra information is available from Harvest Healthcare, just phone 01709 377172 or email [sales@hhLtd.biz](mailto:sales@hhLtd.biz) and we will be happy to help with your query.

