

Data File

Reusable Transfer sheet

Product: LP012 low friction reusable patient transfer sheet.

Fabric: 120gsm reusable Transtex™ RS fabric.

Transtex™ : Cromptons special silicone coated low friction fabric.

Construction: A plain transfer sheet hemmed on all four sides without handles for economy.

Usage: This is a full body length transfer sheet. Recommended usage's include patient turning, repositioning and lateral transfers. It can be used as a pair or as a single sheet folded in half.

Using an approved manual handling technique pass the *Ultra-glide* beneath the patient. Safely position the patient centrally on the slide. Whilst maintaining the correct posture take a firm grasp of the transfer sheet fabric and slide the patient to the desired new position.

Safe patient handling practices should always be adopted by following your local hospital patient handling guidelines. These will be available from your hospital Health & Safety, or, Patient Moving and Handling Co-ordinator.

Colours: Blue – 120gsm Transtex™ RS

Capacity: Maximum patient weight 400kg, (62 stone).
The Ultra-glide transfer sheet should always be inspected prior to use for stitching or fabric degradation which will take place as normal wear & tear and which over a extended period will progressively reduce weight capacity of the transfer sheet.

Warning: Ultra-glide transfer sheets are not intended for lifting. All patient repositioning manoeuvres should be accomplished without lifting.

Laundering: Follow standard hospital laundry guidelines HSG(95)18. The use of bleaching agents should be avoided.
Maximum recommended washing temperature 80° C.
Drip dry, (if necessary a low temperature tumble dry may be used).

Latex free: Yes



CE marking: Cromptons Transtex™ slide and transfer sheets are CE marked to show compliance with the Medical Devices Directive.

