

Dräger DrugTest® 5000 Analyzer

The Draeger DrugTest® 5000 system is a fast, accurate means of testing oral fluid samples for drugs of abuse, such as amphetamines, designer amphetamines, opiates, cocaine and metabolites, benzodiazepines and cannabinoids, as well as precise diagnostic evaluation and data management.



The Draeger DrugTest® 5000 system is comprised of two main components: the Draeger DrugTest® 5000 Test Kits and the Draeger DrugTest® 5000 Analyzer. Reliable and easy to use, the Dräger DrugTest® 5000 Analyzer is a mobile oral fluid screening device for providing qualitative detection of drugs using the Dräger DrugTest® 5000 Test-kits.

Providing reliable and precise analysis of the Test Kit sample within just a few minutes, this state-of-the-art opto-electronic system weighs less than 10 pounds. The Dräger DrugTest® 5000 Analyzer has a user-friendly, menu-driven, illuminated display which can be easily read, even at acute angles.

The Analyzer interprets the results from the DrugTest® 5000 Kit, eliminating any possibility of human error in reading the results. The full color display interface either indicates "Positive" or "Negative" for each drug class.

The Dräger DrugTest® 5000 Analyzer, with its simple three-key operation, can also be easily linked to a wide variety of data recording devices such as a PC, Dräger Mobile Printer or barcode scanner.

The Analyzer's integral data memory allows for up to 500 individual measurements to be stored for printing or upload to a PC.

In addition to data management, a built-in self-test capability controls temperature, optics and general operation.

With easily configurable software, the flexible menu can also be quickly configured to meet the needs of different applications.

Combined with a carry bag, this mobile system is a complete "substance abuse monitoring" system for on-the-spot measurements when equipped with a Dräger DrugTest® 5000 Analyzer, Dräger Mobile Printer, keyboard, DrugTest® 5000 Test Kits, breath alcohol testing device, mouthpieces, and system documentation.



Dräger DrugTest® 5000 Analyzer



Carrying Bag with Complete System

TECHNICAL INFORMATION

Supply Voltage	12 V DC (11 to 15 V DC)
Current Consumption	Typically 3 A
Dimensions (W x H x D)	7.9" x 9.8" x 8.7"
Weight	9.9 lbs
Temperature	Operation: 5 to 40 °C (+41 to +104 °F) Storage/Transportation: -20 to +60 °C (-4 to +140 °F)
Moisture Range	5 to 95 %rh, non condensing
Interfaces	Optical IRDA (printer), PS/2, USB Slave
Duration of One Measurement	5 to 10 minutes, depending on type of test
Storage Capacity	500 data sets
Carrying Bag	with Complete System

ORDER INFORMATION

Item	Order No.
Dräger DrugTest® 5000 Analyzer including power supply, connecting cable and instructions for use.	83 19 900
Accessories	
Compact Keypad (PS/2, dimensions approx. 11.1" x 5.2" x 0.9") "QWERTY" English Keyboard Layout	44 15 002
Car Cable 12 V (Connecting Cable from car socket/cigarette lighter to DrugTest® 5000 Analyzer)	83 12 166
Dräger Mobile Printer	83 19 310
Thermal Paper for Dräger Mobile Printer (1 roll)	44 15 520
Carrying Bag for Dräger DrugTest® 5000 Analyzer	83 19 960
Transport Case	83 19 925
USB Connection Cable for Communication with a PC	44 15 001
PC Software Module for Dräger DrugTest® 5000	83 19 977
Bar Code Scanner	AG 02 491

The Analyzer, the Mobile Printer, keyboard and consumable parts such as test kits are all designed for mobile use. An optional transportation box and/or carrying bag is available for the safe transportation of all system components. The keyboard enables information on the test subject and system operator to be entered, allowing a record of this data to be printed out on the spot along with the Dräger DrugTest® 5000 result.



Carrying Bag with Complete System



Transportation Box



Compact Keyboard



Dräger Mobile Printer

www.draeger.com**USA**

Draeger Safety Diagnostics, Inc.
4040 West Royal Lane, Suite 136
Irving, TX 75063

USA

Tel +1 972 929-1100
Fax +1 972 929-1105

info@dstdi.biz

CANADA

Draeger Safety Canada Ltd.
7555 Danbro Crescent
Mississauga, ON L5N 6P9
Canada

Tel +1 905 821-8988
Fax +1 905 821-2565

sales.canada@draeger.com

MEXICO

Draeger Safety S.A. de C.V.
Av. Pe-uelas No. 5
Bodega No. 37
Fraccionamiento Industrial
San Pedrito
Queretaro, Qro C.P. 76148
Mexico

Tel +1 52 442 246 1113
Fax +1 52 442 246 1114
ventas.mexico@draeger.com

GERMANY

Dräger Safety AG & Co. KGaA
Revalstrasse 1
23560 Lübeck
Germany

Customer Service
Tel +49 451 882-0
Fax +49 451 882-2080