

INSTALLATION MANUAL FOR ROTAID PLUS

1. Determine the location where you wish to mount the Rotaid. Make sure that no irregularities are present at the desired spot on the wall. The preferred height of the cabinet is 1.60 meters (center of cabinet). Please take the fire safety requirements at the specific locations into account. If mounted on a stone or concrete wall, please use the wall anchors.



2. Attach the back plate to the wall via the 4 long screws and rings. On the top an optional 5th mounting position is available if needed.



3. With the back plate secured to the wall, use an Allen key (5 mm) to attach the belt to the back plate. Make sure the nylon belt is firmly attached.



4. After attaching the belt, please use both bolts to find the perfect holding position for your AED device. Make sure both bolts are firmly attached into the aluminum block.



5. You can now rotate the cover on the back plate. Place the hood slightly counter-clockwise on the back wall and turn clockwise as far as possible. When correctly placed, the hook on the bottom of the cabinet falls through the slot in the cover.



6. Your Rotaid Plus Cabinet is ready for usage!

