

SKI SLED ^{CE}

(Part of the “SKI” range of products)

The Ski Sled has been designed for the evacuation of non-ambulant individuals in areas where space is limited, such as congested hospital wards and buildings with narrow corridors or single width fire doors.

Features:

- The tough, slippery base enables individuals to be moved easily over virtually any interior floor surface.
- The foam pad and the internal nylon support struts give protection to the individual while being moved.
- Large foot pocket.
- 3 seatbelt style buckles (metal) and webbing straps.
- Wall mountable, wipe clean bag

Using the SKI Sled:

- In an Emergency the Ski Sled is quickly removed from its wall mountable storage bag and taken to the individual.
- The patient is then transferred on to the Ski Sled (this can be performed either from bed/ floor/ chair).
- Place feet within the foot pocket.
- If pillows are available place these on the individuals leg and chest area (for comfort once strapped in).
- Secure the individual in place on the pad using the seat belt style cross straps.
- Using the end pulling handles the individual can be moved to a safe area. When going down stairs it is important to pull the individual feet first.



Specifications:

Packed: 78cm x 74cm x 13cm

Unrolled: 204cm x 74cm

Padding: 3cm thick Weight: 5kg

Safety tested to: 240kg

Safe working load: 120kg - two people

Material: Thick blue nylon

Product code: SKISL01



Product demonstrations and training packages are available contact us for more details.



www.spectrumhealthcare.co.uk

Email: sales@spectrumhealthcare.co.uk

Telephone: +44 (0)1793 762928

1 Kingsthorpe Business Centre, 63 Studland Road, Kingsthorpe, Northampton, NN2 6NE, UK.
Spectrum Healthcare (UK) Ltd - Registered in England No. 06197553

