

MII

Transtex™ Evacuation Sheet

Product: Patient Evacuation Sheet

Usage: Use for emergency evacuation.

Description: The Transtex™ Evacuation sheet is designed to remain underneath the mattress ready for rapid ward evacuation. The sheet is retained under the mattress in a permanently ready state. In an emergency the two 50mm securing straps are withdrawn from their storage packets and tightened around the patient and mattress, cocooning the patient within the mattress forming a protective sledge. Drag loops provided at each end of the sheet are used to tow and guide the patient from the building.

Fastening type: CS650 Elastic - is held in place by two elastic straps at the mattress head and foot.

Fabric: 120gsm nylon

Colour: Red

Codes: CS650-0001 - elastic attachment CS660-0002 - Velcro strap attachment

Dimension: 185cm x 85cm

Capacity: Maximum patient weight 400kg. Safe working Load 160kg
In normal use your Transtex™ Evacuation sheet does not require any maintenance. Periodic inspection is recommended periodically as accidental damage may have occurred. Attention should be given to stitching, webbing and fabric condition as degradation is possible over an extended period.

Laundering: Follow standard hospital laundry guidelines HSG (95)18. The use of bleaching agents should be avoided.

Latex free: Maximum recommended washing temperature 80° C. Drip dry, (if necessary a low temperature tumble dry may be used).

