

Technical Data

Evacuation Sheet



TM Ref:

EVS

Description

Our Evacuation Sheet fixes semi permanently to the bottom of a flexible foam mattress by means of strong elastics. It has safety belts to hold the patient on the mattress and pulling straps at each end. The combination of the slippery sheet and mattress which will flex will pass through narrow doorways and down stairs. The holding strap at the head end is to hold back on stairways, Two people required.

Dimensions

Size Single bed

Materials Used

Webbing	50 mm in width
Breaking strength	26.5 kN min or 2.65 tonnes min
Flame retardant	Conforms to EEC directive 77/541 (FMV302) burning rate not exceeding 100 mm / min
User weight	Determined by the breaking strength of the webbing at 2.65 tonnes
Red Nylon Fabric	
Tear strength	warp 45N min weft 40N min

Buckles 50 mm Side Release Buckle (plastic)

Care instructions

For general cleaning purposes, wipe the cover with warm water containing a neutral detergent. Where disinfection is required following contamination, refer to hospital disinfection policy. Do not use phenolic based cleaners or bleach on this product .If laundering is necessary and disinfection is required wash up to at 70°C. Tumble dry at low temperature. Not iron.



Wash 70°, Do not bleach, Tumble dry on low temp, Do not iron

Unit 1a Aireside Business Park, Royd Ings Ave Keighley BD21 4BZ

T: 01535 682 630 F: 01535 682 639 E:sales@thorpemill.co.uk www.thorpemill.co.uk

Reg: 1502793